




∞ Meta



AI GLASSES, NOW WITH A FULL RANGE OF LENSES FOR SUPERIOR AESTHETICS AND VISUAL CLARITY.

Available as complete pair or edged lens.

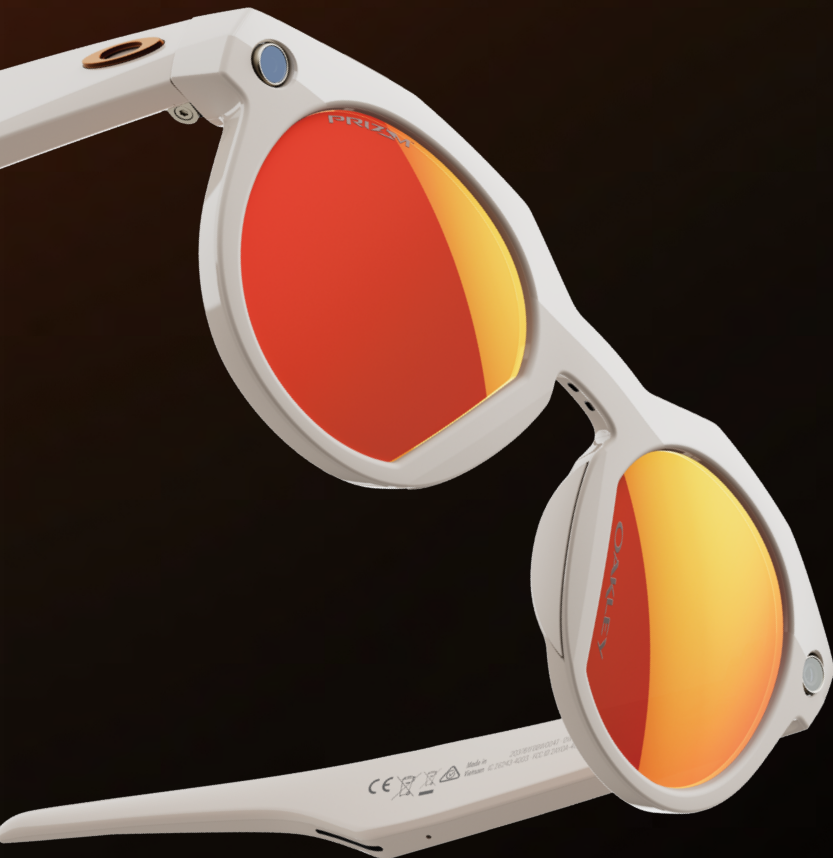


- THE LENSES SHOULD HAVE A FRONT STEP BEVEL WHICH CAN LOOK DIFFERENT WITH EACH PRESCRIPTION.
- ON THINNER PORTIONS OF SOME LENSES, THE STEP BEVEL WILL NOT BE COMPLETE. THIS IS NORMAL AND SHOULD NOT BE APPARENT ONCE MOUNTED.
- ONLY USE SOLVENTS ON LENSES BEFORE MOUNTING.
- ENSURE LENSES ARE COMPLETELY DRY BEFORE PROCEEDING.
- DO NOT HEAT THE FRAME TO REMOVE THE DEMO LENSES OR TO INSERT PRESCRIPTION LENSES. COLD MOUNT ONLY.



∞ Meta

MOUNTING BEST PRACTICES

Mount the lenses from the front using the standard plastic frame mounting technique.

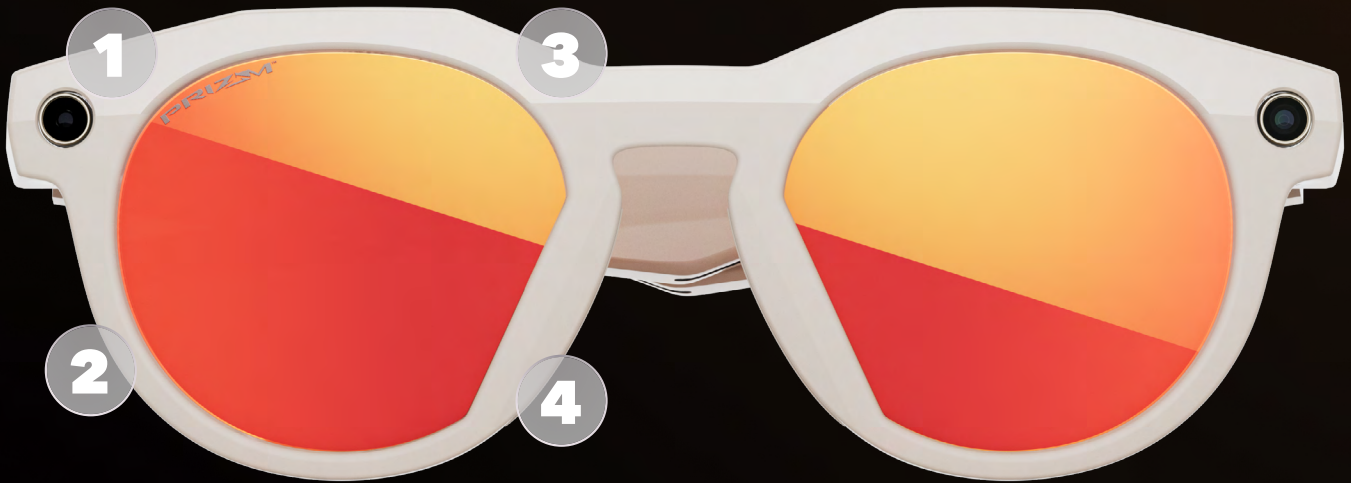


STEP 1



BEGIN BY PLACING **CORNERS 1 AND 2** FIRMLY INTO THE BEZEL.

STEP 2

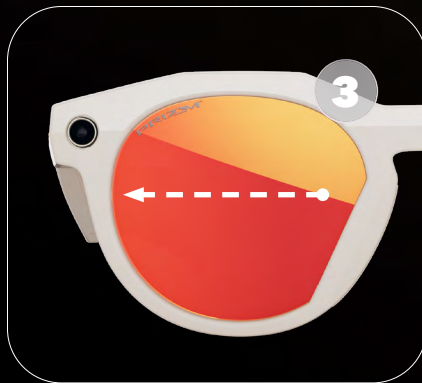

INSERT **CORNER 3** BY APPLYING PRESSURE TOWARDS **CORNERS 1 AND 2** (SEE ARROW), THEN PRESS DOWN INTO **CORNER 3** TO SNAP INTO PLACE.

STEP 3



SNAP IN **CORNER 4** TO COMPLETE MOUNTING

The Oakley Meta warranty is not impacted by the addition of prescription lenses when the proper mounting process is followed.